## Gebruichsanweisung els paintart

### DE | Deutsch

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## Instruction for use els paintart

### EN | English

**1. Product description**  
Light-curing, low viscous color resin for the characterization of restorations in aesthetic dentistry. Available in the colors incisal, white, yellow, blue, red, brown and grey.

**2. Composition**  
BisGMA, BisEMA, highly dispersed silicium dioxide, silanized, catalysts, inhibitors, color pigments

**3. Intended Use**  
Saremco restoratives are intended for the direct reconstruction or correction of functionally compromised natural dentition (e.g., deficient teeth). Restorative material intended for the reconstruction or correction of natural dentition.

**4. Indication**  
Color characterization of composite restorations.

**5. Contra-indication**  
Known allergy to methacrylates, camphorquinone and BHT.

**6. Patient target group**  
els paintart is suitable for use in all patients without any age or gender restrictions.

**7. User**  
els paintart should only be used by a professionally trained dental practitioner.

**8. Side effects**  
In individual cases, contact allergies have been described to products of a similar composition. **Note: els paintart** does not contain TEGDMA or HEMA.

**9. Interactions**  
None known.

**10. Processing stages**

**10.1. Application**  
Attach the application needle. Apply the product directly from the syringe or with a brush onto the cured or uncured composite filling (SAREMCO recommends **apt composite** and **els composite**) and light cure.

**Warning:** Remove the application cannula after each use and replace the cap. Protect light-curing products from strong sources of light.

The specified exposure times refer to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm<sup>2</sup> and a wavelength of 400 - 500 nm. They apply to a maximal layer thickness of 0.5 mm and for all colors. The necessary exposure time may vary depending on the light source and its instructions for use. If in doubt, check the lamp's light output and the necessary exposure time before operation in vitro.

**Exposure time for increments of 0.5 mm**

light power	≥ 500 mW/cm <sup>2</sup>	≥ 1'200 mW/cm <sup>2</sup>
els paintart all colors	40 sec	20 sec

**10.2. Polishing**  
High gloss polishing with polishing brushes, polishing discs, strips or silicone polishers.

**11. Storage**  
Protect light-curing products from strong sources of light and heat! **els paintart** was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Bring to room temperature before use.

**12. Batch number and expiry date**  
The batch number should be specified to identify products in case of enquiries. Products should no longer be used once the expiry date has elapsed.

**13. Precautionary measures**  
Close containers after each use with the right lid. Keep out of reach of children. For dental use only. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice. When polishing or removing composites, it is recommended to always use a water-cooling system and a good extraction system, to ventilate the dental laboratory frequently and to wear masks with high particle filtration efficiency for small particle sizes.

**14. Emergency measures**  
In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an ophthalmologist.

**15. Hygiene**  
Use application instruments for one patient only. Dose products away from patients to avoid contamination. Sterilization of syringes or tips before first use is not required.

**16. Warranty**  
The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether

they are suitable for the intended purpose. We expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental AG.

**17. Other notes for Europe**  
If the user and/or patient become aware of serious incidents connected with application the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides. The Summary of Safety and Clinical Performance of SAREMCO Products can be found in the European database on medical devices (EUDAMED-<https://ec.europa.eu/tools/eudamed/>).

**18. Production / distribution**  
SAREMCO Dental AG  
Gewerbstrasse 4  
CH-9445 Rebstein / Switzerland  
Tel: +41 (0) 71 775 80 90  
Fax: +41 (0) 71 775 80 99  
info@saremco.ch  
www.saremco.ch

Edited 07-2024 | D600174

Class IIa medical devices  
Class III medical devices (Canada)

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